



Operation.

Buttons

Patient Call= Orange Nurse Button.

Emergency = Red Button.

Assistance = Press Green Button and then Orange Patient Button.

Attendance = Green Button.

Reset / Visit = Press Grey Button.

Socket 1

Is for Assisted Technology – Pressure Mats, Bed Sensor/ Pressure Mats, Chair Sensor/ Pressure Mats, Enuresis Mats and Epilepsy sensors.

Automatic monitoring

Connect the Bed or Chair device to [Socket 1](#).

Place occupant in the bed. After 15 seconds the unit will bleep and is armed.

If the resident leaves the bed for 2 seconds the unit will send an accessory event.

This function can also be used for monitoring other devices

** FLOOR PRESSURE MATS WORK WHEN PLUGGED IN to

[Socket 1](#).

Socket 2

Is for Pear Push lead with Orange Patient Button and clip.

This function can also be used for monitoring other devices.

