

HTM 6500iB Room Unit



iButton operation.

Attending a call – when attending a call, place the iButton on the reader, the unit will beep, remove the iButton, the unit will flash a GREEN light. The system will now 'log in' that iButton as **attending**.

When finished attending, press the **reset** button, the unit will beep and flash a RED light. The system will clear the event and 'log out' that iButton's identity.

Multiple staff can be in attendance at one time, each staff member must log in with their iButton, when finished **attending** the last member of staff to leave press's the **reset** button, and the unit will beep and flash a RED light. The system will then clear the event and 'log out' the attending iButton identities.

Night visits – when doing the night checks, place the iButton on the reader, the unit will beep, remove the iButton and the unit will flash a green light press the Reset button within 3 seconds, the unit will beep and flash a RED light. The system will now record a 'Visit' with that iButton's identity.

Buttons. Orange = Patient Call, Red = Emergency, Yellow = Assistance, Grey = Reset.

Socket 1 Is for the connection of accessory triggers and monitoring devices which can be connected to trigger the following events dependant on the accessory adaptor, Patient Call, Accessory and Automatic monitoring. An example of a Patient Call or an Accessory Call would be with a finger switch or floor Mat. The **Automatic bed/chair occupancy monitoring** function is for applications such as bed/chair occupancy monitoring. Connect the monitoring device to **Socket 1** using the **HTM6021** accessory lead.

Place occupant in the bed/chair. After 15 seconds the unit will beep and is armed. If the resident leaves the bed for 2 seconds the unit will send an accessory event.

This function can also be used for monitoring other devices such as bed weight sensors.

Socket 2 Is for the connection of either a **HTM6017** single button lead with reassurance light or a **HTM6018** two button pear push lead with reassurance light and light switch.

Head Office & Repairs Centre: Medicare Systems Ltd, Jade house, 87-89 Sterte Avenue West, Poole, Dorset BH15 2AL

T: 0800 849 5123 [www: medicare systems.co.uk](http://www.medicare systems.co.uk)