

HTM 6500S Room Unit.



Operation.

Buttons and Call types.

Patient Call= **Orange** Nurse Button.

Emergency = **Red** Button.

Attendance = Press **Green** Button.

Assistance = When in attendance press **Orange** Button as a staff to staff call.

Reset / Visit = Press **Grey** Button.

Accessory Call = activated from socket 1 or a lead fault.

Socket 1 Is for the connection of accessory triggers and monitoring devices which can be connected to trigger the following events dependant on the accessory adaptor, Patient Call, Accessory and automatic monitoring. An example of a Patient Call or an Accessory Call would be with a finger switch or floor Mat.

The **Automatic bed/chair occupancy monitoring** function is for applications such as bed/chair occupancy monitoring. Connect the monitoring device to **Socket 1** using the **HTM6021** accessory lead.

Place occupant in the bed/chair. After 15 seconds the unit will bleep and is armed. If the resident leaves the bed for 2 seconds the unit will send an accessory event.

This function can also be used for monitoring other devices such as bed weight sensors.

Socket 2 Is for the connection of either a **HTM6017** single button lead with reassurance light or a **HTM6018** two button pear push lead with reassurance light and light switch.