

Guide on accessory adaptors for use with assistive technology.

	<p><u>HTM 6021 Bed/Chair Mat Adaptor to create an Accessory Call (Purple Banner on Display Panel)</u></p> <p>Used with either a HTM 6001 or any of the HTM 6500 series equipment within socket 1 to activate an Accessory call for (automatic monitoring) for use with Med 9 & Med 10 Bed and Chair mats.</p> <p>(Additional info, monitors a normally closed connection.)</p>
	<p><u>HTM 6022 Floor Mat adaptors to create an Accessory Call (Purple Banner on Display Panel)</u></p> <p>Used with HTM 6001 or any of the 6500 series equipment within socket 1 to activate an Accessory call from a Med 7 & Med 8 Floor Mat. Can also be used to create an Accessory Call with any other normally open connection.</p> <p>(Additional info, monitors a normally open connection.)</p>
	<p><u>HTM 6024 adaptor to create a Normal Call (Orange Banner on Display Panel)</u></p> <p>Used with HTM 6001 or any of the HTM 6500 series equipment within socket 1 to activate a Normal Call from assistive technology that uses a normally open connection. For example a sensitive Jelly bean switch or blow switch.</p> <p>(Additional info, monitors a normally open connection.)</p>
	<p><u>Using/interfacing Assistive technology.</u></p> <p>All assistive technology requires a mono jack plug to connect and work with our adaptor leads. (Stereo jacks will not work.) Operation of a normally closed connection activates on open circuit/contact. Operation of normally open connection needs to close momentarily to activate the equipment. (Latching equipment will not work.)</p> <p>All assistive technology must provide a 0 volt free connection. Any current or voltage on the contacts will cause irreparable damage to the Medicare equipment.</p>

Head Office & Repairs Centre: Medicare Systems Ltd, Jade house, 87-89 Sterte Avenue West, Poole, Dorset BH15 2AL

T: 0800 849 5123 www.medicareystems.co.uk