

Instructions for using a MED32 air monitored bed mat or MED31 air monitored chair mat.

Supplied in each kit will be a control box, air pipe, bed or chair mat and a lead to connect to socket 1 of the Medicare call point.

How it works

The air-filled pressure mat is placed under the mattress and is connected to the control box. As someone gets out of bed the reduction in air pressure in the pressure mat is detected by the control box. The red light on the control box turns on and the alert is triggered.



Bed or chair pressure mat

Positioned under mattress or seat cushion and works with all types of pressure relieving mattresses



Control Box
Attaches to bed using velcro or protective rubber case



Installation instructions



1 Unroll the mat. When newly opening allow the mat to inflate for at least 20 minutes.



2 Plug in and set the control box to suit the mat being used. Connecting both the air pipe and lead to the Medicare call point.



Using the supplied lead, connect the control box to socket 1 of the Medicare call point.



Set the control switch to bed or chair dependant on which mat you are using.

Sensor socket, connects to the air pipe from the mat. Plug in the black air tube between the mat and the control box by sliding the end firmly over the proud airpipe. Ensure it is not bent or crushed to reduce a clear airflow.

3

Check the batteries are in place within the compartment on the back of the box.

4

Position the control box on the bed or wall using the optional rubber case. It can also be attached with the supplied velcro sticker



The rubber case is supplied with reusable rapstraps which can be attached by feeding them through part of the bed and pulling them until tight



5

Finally position the connected mat either under the mattress or seat cushion.

Head Office & Repairs Centre: Medicare Systems Ltd, Jade house, 87-89 Sterte Avenue West, Poole, Dorset BH15 2AL

T: 0800 849 5123 [www: medicare systems.co.uk](http://www.medicare systems.co.uk)