

User guide for the use of floor mats with the Medicare system.

Product code – Description: -

MED7 – Standard economy floor pressure mat.

MED8 Green – Heavy duty floor pressure mat. (Green in colour.) Now with double snap safety lead.

MED8 Black – Heavy duty floor pressure mat. (Black in colour.) Now with double snap safety lead as shown below: -



Dimensions (Width x Depth)

Metric (94cm x 64cm) Imperial (3' 1" x 2' 1")

Triggers an accessory call when pressure is applied. The floor mat is supplied complete with a HTM 6022 Floor Mat Adapter or built in double snap safety lead with a pre-moulded plug coloured **pink**.

The mat **plugs into socket 1** of a Medicare room unit which is also coloured **pink** or the left-hand side socket on earlier models. The location pin clicks into place when fully inserted.



To activate an alarm, apply pressure to the floor mat.

To deactivate or remove the mat press the reset button on the call point and then remove the adaptor, this requires a slight forward motion of the plug combined with pressing the release pin on the plug.

Taking care of your pressure mat.

- Pressure mats **MUST** be kept flat.
- FOLDING, ROLLING or BENDING the mat **WILL** damage the mat and stop it working.
- When not in use please store the mat flat under the bed if appropriate.
- Clean mats with a damp soap solution taking care not to saturate the mat.

DISCLAIMER

The mats are covered under a 3-month limited warranty. Warranty exclude damage caused by folding, bending, rolling or liquid ingress.

Pressure mats may present a trip hazard so care should be exercised with the positioning of the mat.

Head Office & Repairs Centre: Medicare Systems Ltd, Jade house, 87-89 Sterte Avenue West, Poole, Dorset BH15 2AL

T: 0800 849 5123 [www: medicare systems.co.uk](http://www.medicare systems.co.uk)